



# LIFEPAK CR2 AED

## Initial Setup



1

### Remove LIFEPAK CR2 and Battery\* from Box

*\*Please keep in mind the battery may be shipped in a separate box due to shipping regulations.*

2

### Install Battery



- Once battery\* is installed a self-test will begin which may take up to 2 minutes. Wait until you hear 3 audible beeps then continue.

*\*If you're inserting a battery that was previously installed, you will not hear the 3 audible beeps.*

3

### Check for Green Flashing Light

- **Green** flashing light means your AED is ready for use, remember to check monthly.
- If it's not flashing, ensure electrode tray with pads are securely attached or replace the battery.
- Note, battery warranty is 1-year from the date of manufacturer, although should last 4-years.
- If problem persists, please call our office 808-440-8988.





4

## Check the Speaker

- Open the lid and confirm that voice prompts are audible. If so, close the lid to turn off the AED.
- Do not pull red handle at this time, unless it's in the event of an emergency.



5

## Note Battery Install Date

- On the AED battery sticker
  - Month and year the battery was installed
  - Month and year 4 years from installed date
- On inspection tag

*Important:*

The AED must be inspected monthly to ensure it is ready for use at all times. Monthly inspections include checking that the green light is flashing and the AED is not beeping.

Hawaii Office

📞 808-440-8988

✉ info@aed808.com

West Coast Office

📞 949-990-8055

✉ WestCoast@aed808.com



www.aedinstitute.com